


# GP Remover Spear

In Re-treatment of root canal, old Gutta Percha must be removed.

After using ultrasonic, old Gutta Percha can be soften and then it can be removed by GP Remover Spear.

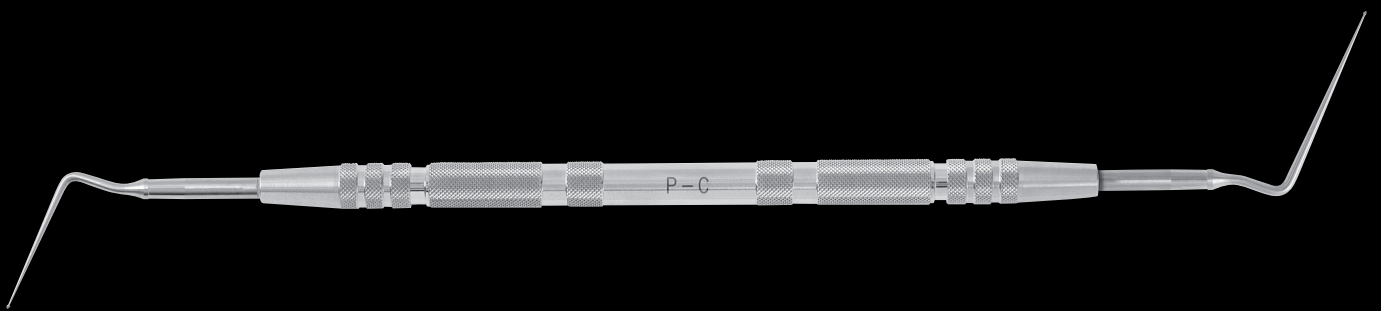






Suitable for  
Scrape off

0.3mm

Suitable for  
Scrape out

0.5mm

0.7mm

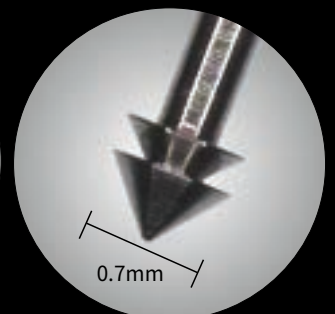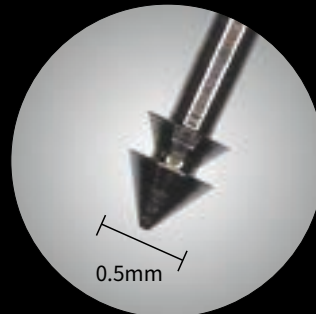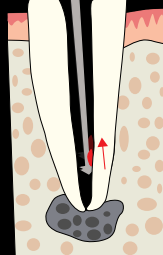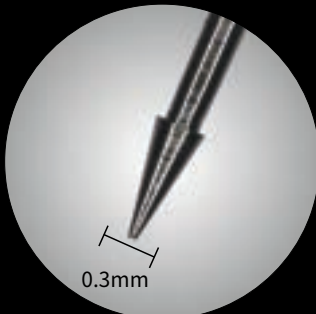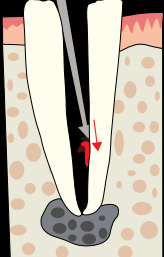

Single

Double

Tips' shape is tiny and have 360degrees of edges,so easy to achieve the target point.  
 It can be used as finalizing remover after using rotary files.

Shall be less power than holding ball-pen.



Example of holdings.

Clinical Movie


Φ 0.3

Φ 0.5


Φ 0.7



Each size can be recognized from marking here.

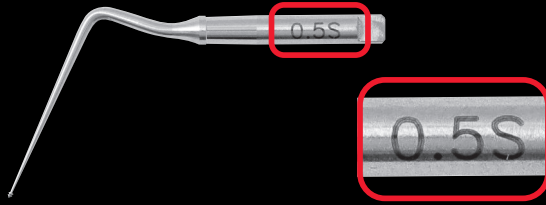

Replaceable tips can be changeable.

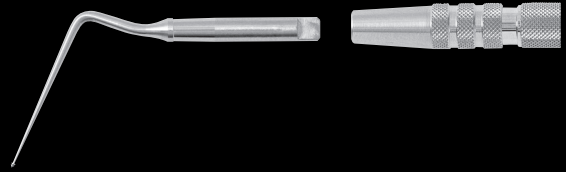

warning




Too much power cause tips bended.



Do not move back bended tips to original shape, which cause breakage of tips.



Micro scope is recommended to be used with.

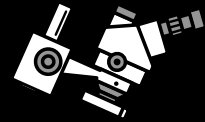


Highly recommended for delicate treatment.

- Tip -

Code	standard	Tip diameter × Length	Tips shape
27131	0.3S	Φ 0.3 × 21mm	Single
27132	0.3L	Φ 0.3 × 28mm	
27133	0.5S	Φ 0.5 × 21mm	Double
27134	0.5L	Φ 0.5 × 28mm	
27135	0.7S	Φ 0.7 × 21mm	
27136	0.7L	Φ 0.7 × 28mm	

Length

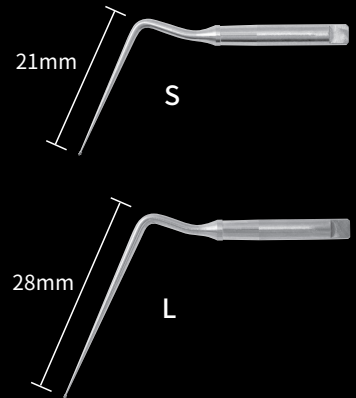

- Handle -

Single End



**P-A** No.10201  
Size/φ8.0mm



**P-F** No.10205  
Size/φ6.0mm

Double End



**P-B** No.10202  
Size/φ8.0mm



**P-C** No.10203  
Size/φ6.0mm

\*Because of YDM's commitment to constant product improvement, YDM reserves the right to change prices and design or discontinue products without a notice.

**YDM CORPORATION**

6-5-20 Tabata, Kita-ku, Tokyo 114-0014, JAPAN  
TEL:+81-3-3828-3161 FAX:+81-3-3827-8991  
e-mail:ydm@ydm.co.jp  
<http://www.ydm.co.jp/en-new/index.html>